Nascar Gel**gel for scar therapy 15 g**

Carefully read the whole instructions for use contained in the package, as it contains important information for the patient about the use of the Nascar Gel, and then store it in case the need to re-read.

Nascar Gel is a medical device.

Nascar Gel is an effective, odorless and safe silicone gel intended for different kinds of scars. Silicones create transparent protective sheathing, which provides proper hydration of the stratum corneum and makes the scar more elastic. They also reduce the size and decoloration of the scar. Silicones eliminate negative feelings of tissue tension and itching around the scar. The proper and early scar care prevents from hypertrophic and keloid scars. It is recommended to use in case of:

- postoperative scars (including scars after mastectomy and caesarean section);
- posttraumatic scars;
- burn scars;
- acne scars;
- hypertrophic and keloid scars;
- scars after removal of tattoos;
- post laser therapy scars;
- stretch marks.

A scar is a skin change, which arises during wound healing. The structure and appearance of scar differs from normal skin. Initially the scar is red, then it becomes less elastic (mature scar), brighter and more shining than surrounding skin. Scarring is natural stage of wound healing but sometimes after injury patients can develop abnormal scars: hypertrophic and keloid scars. The hypertrophic scars are characterized by excessive and pathological development of epithelial tissue around the wound. Scar is thicker and protrudes above the height of normal skin. Keloids appear due to abnormal expansion of scar beyond the wound healing area, so they affect also normal tissue. In both cases scars cause discomfort and unsightly appearance. The early and proper scar care is very important because it allows normal wound healing and prevents pathological changes. Nascar Gel is easily applied to all areas of the skin, including exposed areas like the face and neck; as well as hairy areas without the need for shaving. Nascar Gel is used to soften and flatten scars, relieve the itching and discomfort of scars, and also helps with discoloration of the scars which makes them less visible.

Indications for use. Nascar Gel is used for the prevention and treatment of all types of scars, including keloid and hypertrophic scars. The early and proper scar care is very important because it allows normal wound healing and prevents pathological changes. Nascar Gel is intended for all kinds of scar (including face scars) to:

- make the scar more elastic;
- reduce size and soothe the scar;
- reduce the scar discoloration;
- reduce tissue tension around the scar;
- reduce itching around the scar;
- prevent hypertrophic and keloid scars occurrence.

In order to ensure best results it is recommended to start the use of Nascar Gel as soon as possible after the period of wound epithelialization. Nascar Gel is intended for both adults and children at different age. Before using Nascar Gel pregnant and breast feeding women should consult with their doctor.

Directions for use. For external use only. Nascar Gel should be used according to instructions for use. Ensure that the scar is clean and dry. Wash your hands thoroughly with soap before applying the gel. Apply a thin layer of Nascar Gel to the scar and allow to dry. Should be applied 2 to 4 times a day to exposed areas or as advised by a physician. Small amount of gel massage gently with circular motion onto scar for about 3 minutes, leave until it dries off. In case of applying a large amount of Nascar Gel, gently remove the excess with a dry tissue and allow the drying process to continue. Once dry, gel can be covered by sunscreen, cosmetics, pressure garments or casts. Best apply on fresh scars after healing of wounds, on undamaged skin. During the treatment of scars, avoid exposing the scar to sunlight, high and low temperatures. Also protect the scar from injuries and abrasions. It is inadvisable to use sauna and solarium. In case of any questions, please consult with your doctor, or other qualified health care professional.

Recommended duration of treatment. Apply the Nascar Gel as soon as the wound healed, regularly at least for 2 months.

Precautions. Do not use on open, wet (exuding) and unhealing wounds. Do not use on mucous membrane of eyes, nose and oral cavity. Do not use on earlier applied

preparation with antibiotics or other medicines. Do not use in case of hypersensitivity (allergy) to Nascar Gel ingredients. Do not use after the expiration date printed on packaging. Do not use if the packaging (tube) is damaged. A notice to the user and/or patient: any serious incident that has occurred in relation to the Nascar Gel should be reported to the manufacturer and the competent authority of the marketed countries in which the user and/or patient is established.

Composition. Dimethylsiloxane, cyclopentasiloxane, dimethylsiloxane crosspolymer. Medical device does not contain water and preservatives.

Shelf life. 3 years. Batch number and expiry date is on the seal of the tube and on the box.

Storage conditions. Store in the original container at a temperature below 25°C. Keep out of the reach and sight of children.

Packaging. 15 g in a plastic tube, 1 tube in a carton box with instructions for use. Dispose of in accordance with local regulations.

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